dry-turbo cd 2[®]

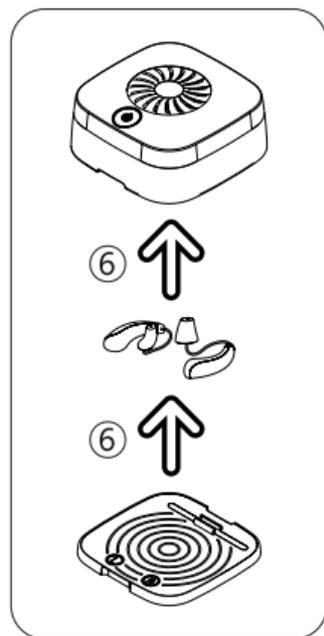
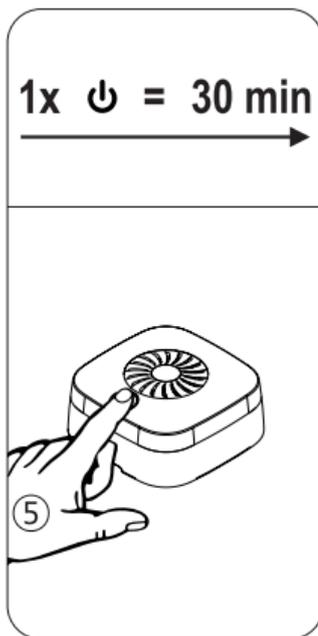
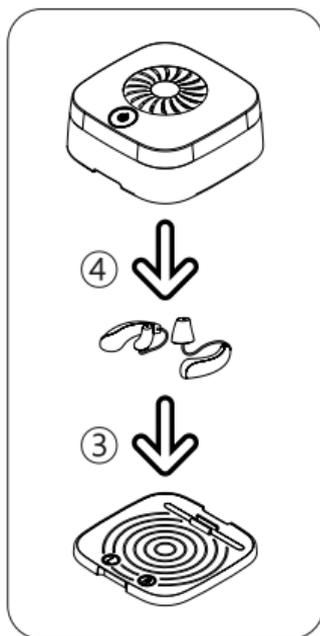
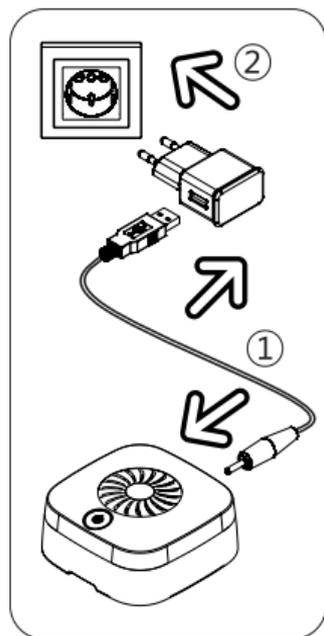
Mini turbo drying cap for all hearing aids.

INSTRUCTION FOR USE

EN INSTRUCTION FOR USE
DE GEBRAUCHSANWEISUNG
FR MODE D'EMPLOI
ES MANUAL DE INSTRUCCIONES
PT MANUAL DE INSTRUÇÕES
IT MANUALE DI ISTRUZIONI

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EN INSTRUCTION FOR USE

Dries and cares your hearing aids

Moisture and cerumen can impair the functioning of hearing aids and cause failures. The dry-turbo cd 2[®] offers optimal care through fast drying by fan and hygiene with UV-light. This clinical cleanliness prevents ear infection risks and protects both your hearing aids and your health.

Using first time

1. Plug the USB cable into a USB socket of the USB power supply and the round plug into the socket on the back of your dry-turbo cd 2[®]
2. Then plug the power supply unit into a mains socket ① The dry-turbo cd 2[®] is ready to use

Operation

3. Place your wet wipe pre-cleaned hearing aids on the Hygiene-Pad 2[®] with the battery compartment open
4. Place the dry-turbo cd 2[®] with the acrylic hood over the Hygiene-Pad 2[®]
5. Press the button and the care program (drying + UV-light) starts automatically
 - The drying process starts, the fast drying by fan is processor-controlled over a period of 30 min ① LED permanent light
 - After 30 min the dry-turbo cd 2[®] automatically shutdown to stand-by ① LED switch off
6. Lift the dry-turbo cd 2[®] up and off next to the Hygiene-Pad 2[®], remove the dried hearing aids

Maintenance and handling

We recommend wet wipes for cleaning your dry-turbo cd 2[®]. Please also read the care and operating instructions for your hearing aids. If you press the button for longer than 3 seconds during the drying phase, you can switch off the device manually. The dry-turbo cd 2[®] is very energy efficient; unplug the power supply only for prolonged absence.

Warnings

Avoid contact with water. Don't use the dry-turbo cd 2[®] near basins etc. In the event of a malfunction, unplug the power supply and take the dry-turbo cd 2[®] to your retailer. Don't disassemble the dry-turbo cd 2[®] and use only the supplied 5V power adapter. Children must be supervised to ensure that they don't use the dry-turbo cd 2[®] as a toy. The dry-turbo cd 2[®] is not to be used by persons who have limited physical, sensory or mental abilities or who do not have sufficient experience and/or knowledge of the dry-turbo cd 2[®], unless they are supervised for their own safety by a responsible person or receive instructions on how to use the dry-turbo cd 2[®]. In idle mode, you must then restart the device.

Do not dispose of electrical appliances as unsorted municipal waste; use separate collection facilities. If electrical appliances are disposed of in landfills or dumps, hazardous substances can leak into the groundwater and get into the food chain, damaging your health and well-being.

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© 2010-2021 flow-med GmbH. All rights reserved.
flow-med GmbH