



dry-turbo cd 2.1[®]

Mini turbo drying cap for all hearing aids.

INSTRUCTION FOR USE

EN	INSTRUCTION FOR USE	2 - 3
DE	GEBRAUCHSANWEISUNG	4 - 5
FR	MODE D'EMPLOI	6 - 7
ES	MANUAL DE INSTRUCCIONES	8 - 9
PT	MANUAL DE INSTRUÇÕES	10 - 11
IT	MANUALE DI ISTRUZIONI	12 - 13
NL	GEbruIKSAANWIJZING	14 - 15
GR	ΕΓΧΕΙΡΙΔΙΟ ΟΔΗΓΙΩΝ	16 - 17
TR	KULLANIM KILAVUZU	18 - 19
DK	BRUGERVEJLEDNING	20 - 21
SE	BRUKSANVISNING	22 - 23
NO	BRUKSANVISNING	24 - 25
PL	INSTRUKCJA OBSŁUGI	26 - 27
CN	用法說明	28 - 29
JP	取扱説明書	30 - 31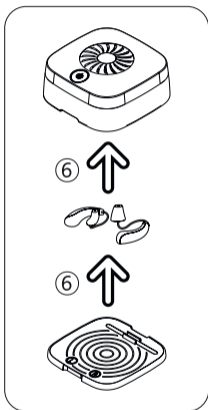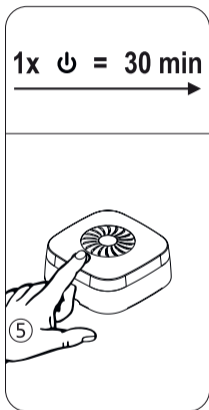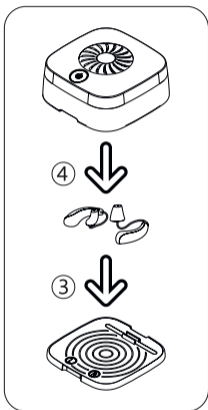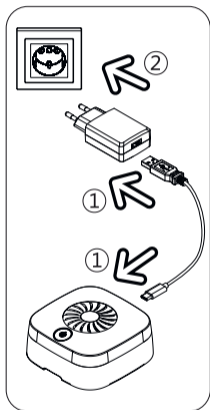

EN INSTRUCTION FOR USE

Dries and cares your hearing aids

Moisture and cerumen can impair the functioning of hearing aids and cause failures. The dry-turbo cd 2.1® offers optimum care due to fast drying by fan and hygiene due to UV-light with antimicrobial effect. This prevents the risk of ear infections and protects both your hearing aids and your health.

Using first time

1. Insert the USB-cable into the 5 V power supply and the USB-C connector into the jack on the rear side of your drying station
2. Then plug the power supply unit into a mains socket
 - ① The dry-turbo cd 2.1® is ready to use

Operation

3. Place your wet wipe pre-cleaned hearing aids on the Hygiene-Pad 2® with the battery compartment open
4. Place the dry-turbo cd 2.1® with the acrylic hood over the Hygiene-Pad 2®
5. Press the button and the care program (drying + UV-light) starts automatically
 - The drying process starts, the fast drying by fan is processor-controlled over a period of 30 min ① LED permanent light
 - After 30 min the dry-turbo cd 2.1® automatically shutdown to stand-by ① LED switch off
6. Lift the dry-turbo cd 2.1® up and off next to the Hygiene-Pad 2®, remove the dried hearing aids

Maintenance and handling

We recommend wet wipes for cleaning your dry-turbo cd 2.1®. Please also read the care and operating instructions for your hearing aids. If you press the button for longer than 3 seconds during the drying phase, you can switch off the device manually. The dry-turbo cd 2.1® is very energy efficient; unplug the power supply only for prolonged absence.

Warnings

Avoid contact with water. Don't use the dry-turbo cd 2.1® near basins etc. In the event of a malfunction, unplug the power supply and take the dry-turbo cd 2.1® to your retailer. Don't disassemble the dry-turbo cd 2.1® and use only the supplied 5V power adapter. Children must be supervised to ensure that they don't use the dry-turbo cd 2.1® as a toy. The dry-turbo cd 2.1® is not to be used by persons who have limited physical, sensory or mental abilities or who do not have sufficient experience and/or knowledge of the dry-turbo cd 2.1®, unless they are supervised for their own safety by a responsible person or receive instructions on how to use the dry-turbo cd 2.1®. In idle mode, you must then restart the device.

Do not dispose of electrical appliances as unsorted municipal waste; use separate collection facilities. If electrical appliances are disposed of in landfills or dumps, hazardous substances can leak into the groundwater and get into the food chain, damaging your health and well-being.

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© 2010-2024 flow-med GmbH. All rights reserved.
flow-med GmbH
Otoplastic AG