



# U-sonic 3

Rechargeable mini ultrasonic cleaner.

INSTRUCTION FOR USE

EN	INSTRUCTION FOR USE	2 - 3
DE	GEBRAUCHSANWEISUNG	4 - 5
FR	MANUEL D'UTILISATION	6 - 7
ES	MANUAL DE INSTRUCCIONES	8 - 9
PT	MANUAL DE INSTRUÇÕES	10 - 11
IT	MANUALE DI ISTRUZIONI	12 - 13
NL	GEBRUIKSAANWIJZING	14 - 15
TR	KULLANIM KILAVUZU	16 - 17
DK	BRUGERVEJLEDNING	18 - 19
FI	KÄYTTÖOPAS	20 - 21
SE	ANVÄNDARHANDBOK	22 - 23
NO	BRUKSANVISNING	24 - 25
HU	HASZNÁLATI ÚTMUTATÓ	26 - 27
PL	INSTRUKCJA OBSŁUGI	28 - 29
CZ	UŽIVATELSKÁ PŘÍRUČKA	30 - 31
SL	NÁVOD NA POUŽITIE	32 - 33
CN	用法說明	34 - 35
JP	取扱説明書	36 - 37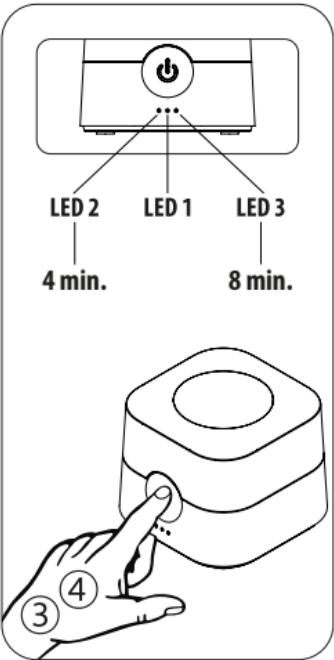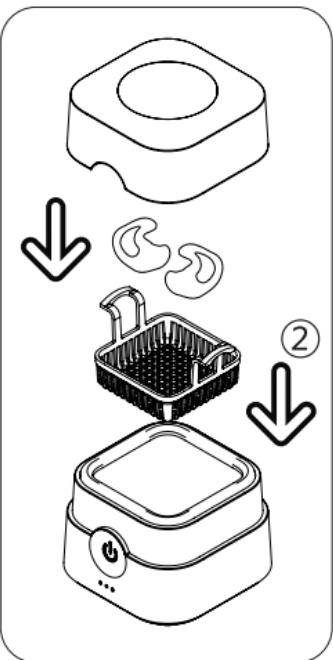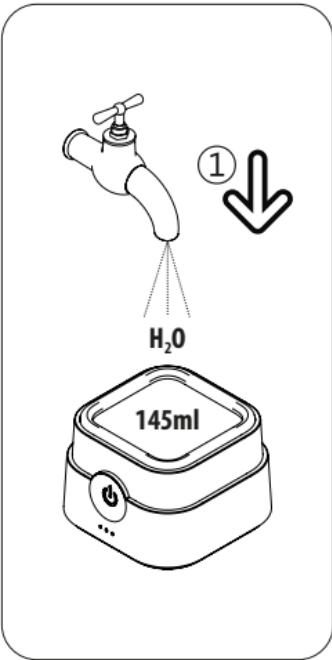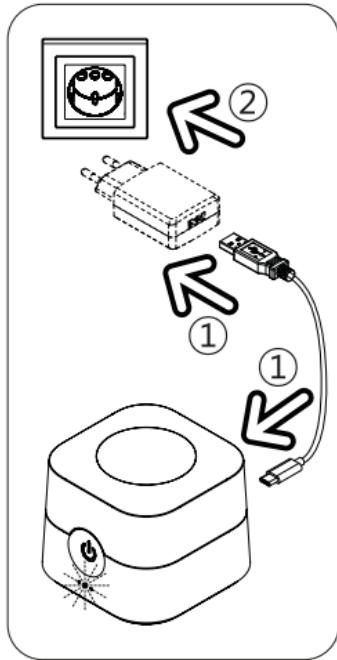

# EN INSTRUCTION FOR USE

## Perfect cleaning and care with ultrasound

Especially for small products, germ-free, deep cleaning is important for daily use, personal hygiene and value retention. Powerful ultrasound at 42 kHz ensures hygienic cleanliness even with clear water.

### Initial operation

1. Plug the USB cable into a USB power supply (5V/2A) and the USB-C plug into the socket on the rear of your u-sonic 3.  
The u-sonic 3 is charged in approx. 3 hours and is ready for operation for approx. 7 short cleanings  
 LED 1 flashes green.
2. Then plug the power supply unit into a mains socket.  
 LED 1 lights up green

### Operation

1. Fill the stainless steel tub of the u-sonic 3 with water up to the mark. You can also use the lid for this purpose.
2. Place the parts to be cleaned in the cleaning basket or directly into the water and close the lid.
3. Touch the sensor button 1x briefly and the ultrasonic cleaning starts  
in the short program 4 minutes.  
 LED 2 display flashes
4. Touch the sensor button 2x briefly and the ultrasonic cleaning starts  
in the intensive program 8 minutes.  
 LED 3 indicator flashes
5. After ultrasonic cleaning is completed, the unit automatically switches off in stand-by mode.  
 all LEDs switch off
6. Open the lid and remove the cleaned parts
7. After each cleaning, empty the water into a drain basin

## **Maintenance and care**

- Switch off the unit and disconnect the power plug before each cleaning and maintenance.
- After ultrasonic cleaning, rinse the cleaned parts with running water. Tubes and holes must be blown through well and dried.
- Simple cleaning is done with clear water. For intensive cleaning, a cleaning concentrate is recommended as an additive or a splash of mild detergent.
- If you touch the sensor button for longer than 3 seconds during ultrasonic cleaning, you can manually switch the unit to stand-by.
- To clean your u-sonic 3, we recommend wiping it with cleaning damp cloths.

## **Application examples**

Earplugs, swimming and sleeping protection, mini tubes, ear olives and domes, coins, medals, badges, razor shaver heads, drawing pens, dental prostheses, braces, cosmetic utensils, infant teats, nipples, ear jewelry, necklaces, rings and other jewelry items that are ultrasonically cleanable; ask your jeweler!

## **! Warnings**

Never place electronic devices in the liquid! Only ultrasonic cleanable parts may be cleaned in the u-sonic 3! The u-sonic 3 must not be operated near washbasins or similar - risk of electric shock! Never operate the u-sonic 3 without water in the stainless-steel tank! In case of malfunctions, please disconnect the power plug and take the unit to the dealer. Do not disassemble the unit. Avoid mechanical stress due to impact and falling! The u-sonic 3 must not be operated near fire or easily combustible objects! Batteries do not belong in the hands of children. Do not heat the battery above 60 °C (140°F) and do not throw it into fire: risk of explosion and fire! Do not short-circuit the battery! Never discharge the battery completely, this will shorten its lifetime. If the battery is to be stored for a longer period of time, a residual capacity of approx. 30% is recommended. This device is not intended for use by persons with reduced physical, sensory or mental capabilities, or lack of experience and/or knowledge, unless they have been given supervision or instruction concerning use of the device by a person responsible for their safety! Follow the instructions on Waste Electrical and Electronic Equipment (WEEE) on page 38.

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© 2010-2023 flow-med GmbH. All rights reserved.
**flow-med GmbH**
Otoplastic AG